



3.0mmD Tapered Implant Body



3.0mmD Tapered Implant Body drill sequence components:

- Four implant body views (top and bottom views).
- SD2.3 2.3/2.0mmD (Soft Bone Final)
- SD2.8 2.8/2.3mmD (Dense Bone Final)
- CBD3.2 (Optional Bone Contour)


3.7mmD Tapered Implant Body



3.7mmD Tapered Implant Body drill sequence components:

- Seven implant body views (top and bottom views).
- SD2.3 2.3/2.0mmD
- SD2.8 2.8/2.3mmD (Soft Bone Final)
- SD3.4 3.4/2.8mmD (Dense Bone Final)
- CBD3.7 (Optional Bone Contour)


4.7mmD Tapered Implant Body

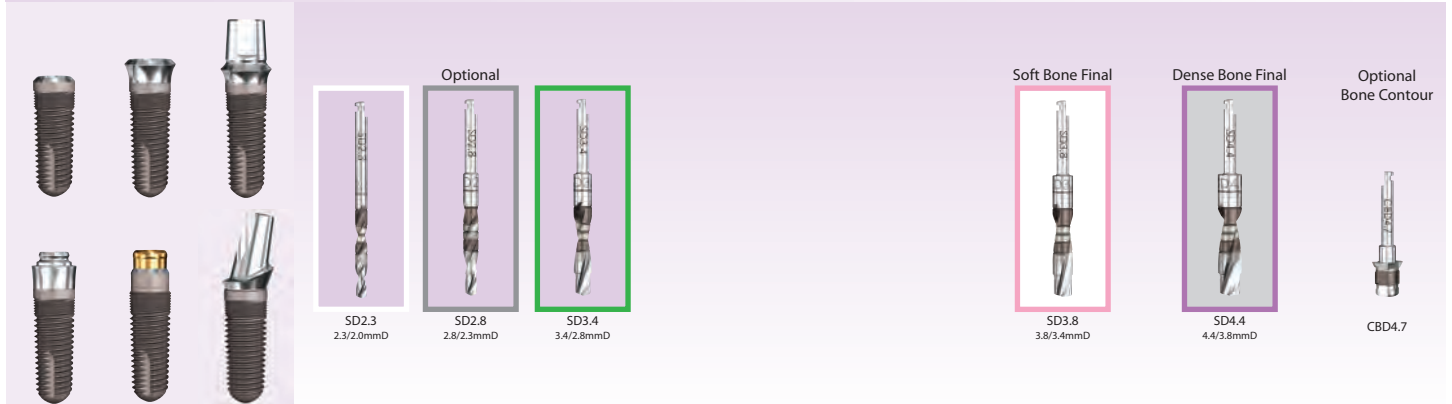

4.7mmD Tapered Implant Body drill sequence components:

- Seven implant body views (top and bottom views).
- SD2.3 2.3/2.0mmD
- SD2.8 2.8/2.3mmD (Optional)
- SD3.4 3.4/2.8mmD (Dense Bone Final)
- SD3.8 3.8/3.4mmD (Soft Bone Final)
- SD4.4 4.4/3.8mmD (Dense Bone Final)
- CBD4.7 (Optional Bone Contour)

5.7mmD Tapered Implant Body

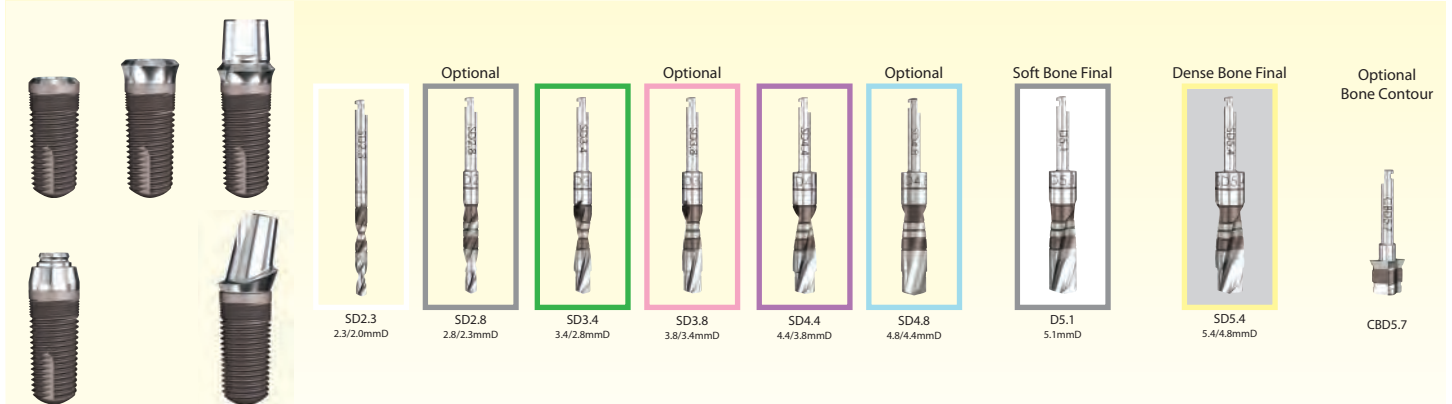

5.7mmD Tapered Implant Body drill sequence components:

- Seven implant body views (top and bottom views).
- SD2.3 2.3/2.0mmD
- SD2.8 2.8/2.3mmD (Optional)
- SD3.4 3.4/2.8mmD (Dense Bone Final)
- SD3.8 3.8/3.4mmD (Optional)
- SD4.4 4.4/3.8mmD (Optional)
- SD4.8 4.8/4.4mmD (Optional)
- D5.1 5.1mmD (Soft Bone Final)
- SD5.4 5.4/4.8mmD (Dense Bone Final)
- CBD5.7 (Optional Bone Contour)